

CA19-9, FLUID

Orderable - CA199F

Turnaround Time: 4 hours

Specimen:

Fluid

Collection Information:

Collect a minimum of 1 mL of fluid in a sterile tube without additive (e.g. orange top urine container). Grossly hemolyzed samples are unacceptable.

May also wish to collect a Light Green top (Li-Heparin) Vacutainer tube of blood to compare fluid CA19-9 result to plasma CA19-9 result.

Reference Ranges:

No reference range available for fluid

Interpretive Comments:

It may be helpful to interpret fluid CA19-9 result relative to plasma CA19-9 result.

Specimen Stability:

Stable for 14 days at 2-8 °C
Stable for 3 months at -20 °C



Laboratory:
Core Lab



Requisition:
GENERAL LABORATORY
REQUISITION



Method of Analysis:
Roche
Electrochemiluminescence



Test Schedule:
As required