

CLONAZEPAM,
SERUM

Orderable - CLON

Turnaround Time: 5 days

Alternate Name(s):

Rivotril

Specimen:

2 x 6 mL Red top Vacutainer tube

or

2 x 4.5 mL Green (Lithium Heparin) top Vacutainer tube

Collection Information:

Collect blood aseptically in a Vacutainer tube.

Minimum volume of serum or plasma required for testing is 3 mL.

Spin and aliquot serum or plasma and freeze immediately before sending to the lab

Reference Ranges:

20-180 nmol/L

Interpretive Comments:

Alert value is 320 nmol/L

Storage and Shipment:

Separate and freeze. Ship frozen and do not allow time to thaw



Laboratory:
Toxicology/Special
Chemistry Lab



Requisition:
GENERAL LABORATORY
REQUISITION



Method of Analysis:
LC-MS/MS



Test Schedule:
Monday – Friday
0800-1600