

COLD AGGLUTININ SCREEN TEST

Orderable - COLDSC

Turn Around Time: 3 hours

STAT: 3 hours

Specimen:

6 mL Pink (EDTA) top Vacutainer tube

Collection Information:

Note: Must be kept at 37°C from time of sample collection. Place sample in a container with warm water (37°C) from time of collection until delivered to Blood Transfusion Laboratory.

Reference Ranges:

See report

Interpretive Comments:

If the cold agglutinin screen is positive, the physician may request the Cold Agglutinin Titre and Thermal Amplitude.

Storage and Shipment:

Transport sample to the lab in a container with warm water (37°C) within 30 minutes of collection.



Laboratory:
Blood Transfusion
Laboratory



Requisition:
[BLOOD TRANSFUSION
REQUISITION](#) or
Electronic order



Method of Analysis:
Saline tube test



Test Schedule:
As required
Monday-Friday