

CORTISOL, 24-HOUR URINE

Orderable – CORU24

Turnaround Time: 7 days

Specimen:

24-hour urine

Collection Information:

[Collection of 24-Hour Urine Sample](#)

Collect urine over a 24-hour period in a container with no preservative.

Please complete the laboratory label attached to the collection bottle to include patient's name, collection start time and date and collection end time and date.

Reference Ranges:

≤ 274 nmol/day

Special Processing:

Transfer a 10-mL aliquot of the measured urine to a sterile container and freeze.



Laboratory:
Core Lab



Requisition:
GENERAL LABORATORY
REQUISITION



Method of Analysis:
IMMULITE



Test Schedule:
Referred out Monday-
Friday



Referred Out Location:
[LifeLabs](#)