

PEMPHIGUS/
PEMPHYGOID
ANTIBODIES

Orderable - ASKIN

Turn Around Time: 8 days

Alternate Name(s):

Anti Skin Antibodies
Skin Antibodies

Specimen:

Adult	Pediatric
5 mL Gold top Vacutainer tube	0-2 years: 0.5 mL Red Microtainer 2-10 years: 2 mL Red top tube

Collection Information:

Collect blood. Allow blood to clot at room temperature for 30 minutes and separate by centrifugation.

Transfer an aliquot of serum to a labelled tube and cap tightly.

Reference Ranges:

Negative

Special Processing:

Grossly hemolyzed, lipemic or microbially contaminated specimens may interfere with the performance and should be documented.

Comments:

Dilutions
1:20 to 1:160



Laboratory:
Core Lab



Requisition:
GENERAL LABORATORY
REQUISITION




Method of Analysis:
Indirect
immunofluorescence



Test Schedule:
Referred out Monday-
Friday



Referred Out Location:
[LifeLabs Medical
Laboratory Service](#)

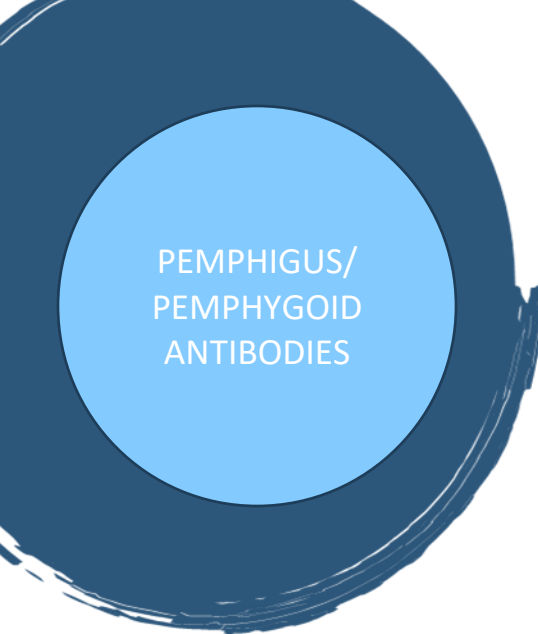

PEMPHIGUS/
PEMPHYGOID
ANTIBODIES

Storage and Shipment:

Intercampus - Ship via routine or stat specimen transport. Serum can be stored for up to 4 days at 2-8°C. For longer storage, store at -20°C.

Do not ship after 1500 h.

Store at 2-8° and ship ASAP (Monday to Friday only).

Intercity - Freeze and store serum samples at -20°C. Ship on ice packs. Avoid freeze/thaw cycles.