

PAROXETINE, SERUM

Orderable - PAROX

Turnaround Time: 7 days

Alternate Name(s):

Paxil

Specimen:

6 mL Red top Vacutainer tube

Avoid blood collection tubes with separator gels (e.g., Gold top).
Light Green (Li-Heparin) top tubes are also **NOT** acceptable.

Collection Information:

Collect blood aseptically in a Vacutainer tube 10 - 14 hours after last dose.

Collect trough specimen prior to first morning dose.

A minimum volume of 0.5 mL of serum is required for testing.

Reference Ranges:

Please note change in units and therapeutic range as of October 31, 2022.

20-65 ng/mL

Storage and Shipment:

Separate, store and send frozen.



Laboratory:
Core Lab



Requisition:
GENERAL LABORATORY
REQUISITION



Method of Analysis:
LC-MS/MS



Test Schedule:
Referred out Monday-
Thursday



Referred Out Location:
[In-Common Laboratories](#)