

GYNAECOLOGICAL  
CONVENTIONAL  
SMEAR FOR  
CYTOLOGY

## Orderable – E-order/Requisition

Turnaround Time: 1 week

### Alternate Name(s):

PAP Smear  
Combined Cervical/Endocervical Smear  
Endocervical Smear  
Vaginal Smear

### Specimen:

Gynaecological: Conventional Smear  
Smear fixed slide

### Collection Information:

Clinical history is an important component for diagnostic interpretation.  
Please check pertinent clinical information on the requisition.

### Reference Ranges:

See report

### Interpretive Comments:

Clinical history is an important component for diagnostic interpretation.  
Please check pertinent clinical information on the requisition.

### Special Processing:

Spray the smeared slide immediately with Cytology spray fixative.  
Allow to dry before placing in the slide transport container.  
Using a pencil label the frosted end of the smeared slide with the patient's name.



**Laboratory:**  
Cytopathology Lab




**Requisition:**  
[CYTOPATHOLOGY](#)  
[REQUISITION- NON-](#)  
[GYNAECOLOGICAL AREA](#)



**Method of Analysis:**



**Test Schedule:**  
Weekdays

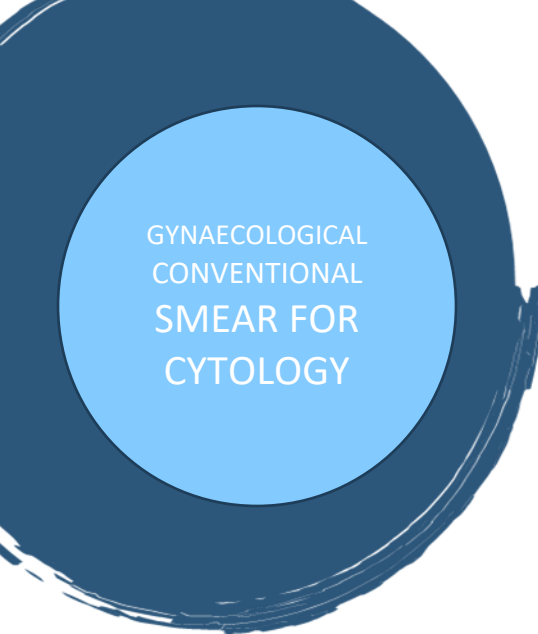

GYNAECOLOGICAL  
CONVENTIONAL  
SMEAR FOR  
CYTOLOGY

**Unlabelled slides cannot be processed.**

**Critical Information Required:**

Specimen slide **must be** spray fixed and labelled with the patient's name.

**Storage and Shipment:**

Place the slide in a cardboard or plastic transport container.

Transport all specimens in a biohazard transport bag with the specimen in the zipped area separating the requisition in the side pocket.