

FARMER'S LUNG IGG ANTIBODIES SERUM

Orderable - AAFL

Turn Around Time: 10 days



Laboratory:
Core Lab



Requisition:
GENERAL LABORATORY
REQUISITION



Method of Analysis:
Fluorescence Enzyme
Immunoassay (FEIA)



Test Schedule:
Referred out Monday-
Thursday



Referred Out Location:
[In-Common Laboratories](#)

Alternate Name(s):

Allergic Alveolitis
Allergic Lung Serology
Farmer's Lung
Farmers Lung
Farmer's Lung Precipitins
Micropolyspora faeni
M. faeni
Laceyella sacchari
L. sacchari
Hypersensitivity Pneumonitis

Specimen:

Adult	Pediatric
5 mL Gold top Vacutainer tube	2-10 years: 3 mL Red top Vacutainer tube

Collection Information:


A single 1 mL aliquot is sufficient to perform this test as well as Aspergillus fumigatus, Avian, Budgie, Pigeon and Parrot IgG Antibodies tests.

To order Allergic Lung Serology please order both Farmer's Lung IgG Antibodies AND Aspergillus fumigatus IgG Antibodies (separate orderable).

Reference Ranges:

M. faeni IgG antibodies:	<5.2 mg/L
L. sacchari IgG Antibodies:	<21.5 mg/L

Storage and Shipment:

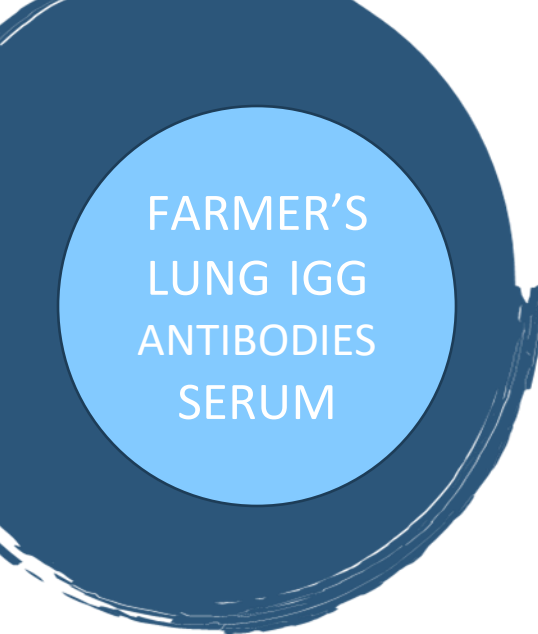

FARMER'S
LUNG IGG
ANTIBODIES
SERUM

Store and ship and serum frozen.

Specimen Stability:

Refrigerated: 7 days

Frozen: 3 months

Comments:

T. vulgaris IgG Antibodies replaced by L, sacchari IgG Antibodies June 2022.