

Disabling Alarms


If you need to disable an alarm, touch the measurement (the number) and disable the alarm.

BEWARE: The alarm ON/OFF is immediately above the “Measurement I/O (ON/OFF)”. If you turn the measurement OFF, no data will be collected.

Turning All Alarms Off


During WLS or a resuscitation where the team is present and the patient is triggering continuous alarms, you may turn the alarm off as follows:

1. Go to Alarm limits (from Smart Keys)

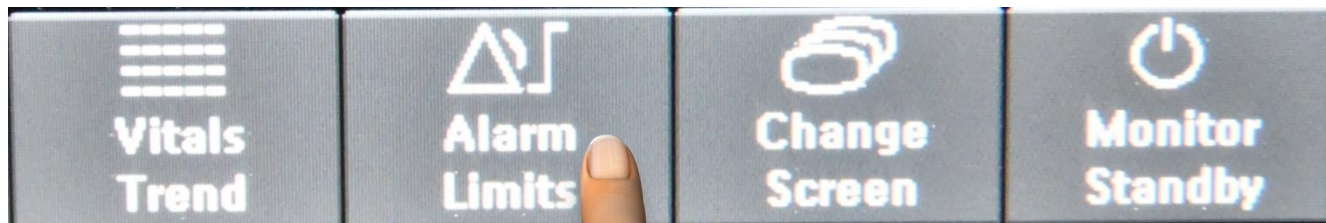

Turning All Alarms OFF

2. Choose “All Alarms Off”.

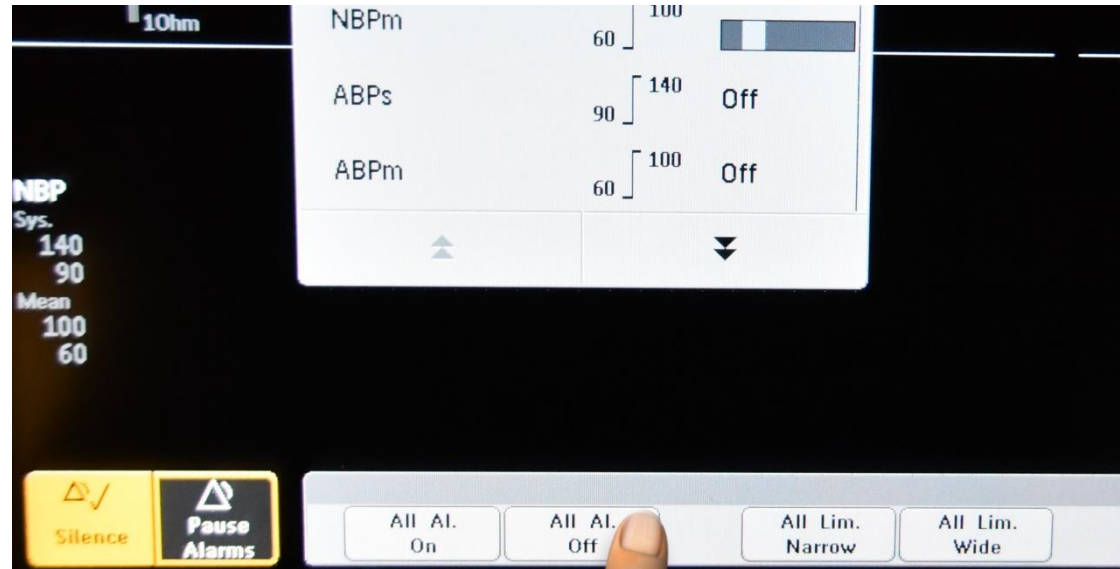

Turning All Alarms OFF

3. Confirm from the menu at the bottom. When you confirm, you are assuming responsibility for the disabled alarms (document the reason the alarms are OFF).

