





Pathology and Laboratory Medicine

Orderable - LASP
Turn Around Time: 10 days

Alternate Name(s):

Asparaginase Activity



Laboratory: Core Lab



4 mL Lavender top Vacutainer tube



Requisition:

Collection Information:

L-ASPARAGINASE REQUISITION

Completed requisition must accompany labeled sample to the lab.

Reference Ranges:

See report

Special Processing:



Test Schedule:

Referred out Monday-

Thursday

SRA:

Centrifuge 10 minutes @ RT.

Aliquot into 2 mL aliquot tube and freeze immediately in -70°F freezer.



Referred Out Location:

CHU Sainte-Justine University of Montreal Cote Saint-Catherine Road Montreal, QC H3T 1C5

514-345-4642

Storage and Shipment:

Store at -70°F. Ship on dry ice.